



Welcome to the eSee Digital Acuity Chart, the easy, accurate, affordable chart for your modern practice.. For in depth setup instructions, user guide, and warranty information, visit www.eseeacuity.com/guides.

Quick Setup

1. Plug the chart into the wall. It will boot to the settings screen.
2. Using the up and down arrows (8,11) to navigate and the left and right arrows (9,10) to change selections, set the system distance (the distance from the chart to the patient's eyes) and whether or not the chart should be mirrored. You can always return to this screen later if necessary.
3. Use the selection button (12) to select Apply.
4. Enjoy!

Note: many customers start by propping the chart on a table or in a chair. This is perfectly fine! However, the IR sensor is behind the monitor, so a particularly plush chair or extreme distance from a wall may affect remote sensitivity.

Feature List

- | | |
|--------------------------------|---|
| Letter chart | Fixation disparity |
| Number chart | Astigmatic dial |
| Tumbling E's | Custom slideshow |
| Picture & symbol chart | Custom education images |
| Up to 10 version of each chart | Custom pediatric images |
| HOTV chart | Mirroring |
| ETDRS chart | Silent |
| Contrast sensitivity | Programmable function buttons |
| Duo-chrome | Works in lanes from 6' to 45' in length |
| Worth 4-dot | No bulbs to replace |
| Muscle light for fixation | Up and running in minutes |

